# User Manual

## **POC-624-12 Series (DC-In Model)**

**23.8" Computer with  
11th Generation Intel®  
Processor**

**ADVANTECH**

*Enabling an Intelligent Planet*

# Instructions for the User

This document combines text and illustrations, providing a comprehensive overview of the system. The information is presented as sequential action steps, allowing the user to directly learn how to use the device.

The text provides explanations and instructs the user step-by-step in the practical use of the product, with short, clear instructions in an easy-to-follow sequence.

## Definitions

**Warning!** A **WARNING** statement provides important information about a potentially hazardous situation which, if not avoided, could result in serious injury or death.

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**Caution!** A **CAUTION** statement provides important information about a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or in damage to the equipment or other property.

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**Note!** A **NOTE** provides additional information intended to avoid inconveniences during operation.

## Safety Instructions

1. Strictly follow these Instructions for use. Please read these safety instructions carefully.
2. Please keep this user manual for later reference; any use of the product requires full understanding and strict observation of all portions of these instructions. Observe all WARNINGS and CAUTIONS as rendered throughout this manual and on labels on the equipment.
3. Repair of the device may also only be carried out by trained service personnel. Advantech recommends that a service contract be obtained with Advantech Service and that all repairs also be carried out by them. Otherwise, the correct functioning of the device may be compromised.

**Warning!** *Because of the danger of electric shock, never remove the cover of a device while it is in operation or connected to a power outlet.*

4. If one of the following situations arises, get the equipment checked by service personnel:
  - The power cord or plug is damaged.
  - Liquid has penetrated into the equipment.
  - The equipment has been exposed to moisture.
  - The equipment does not work well, or you cannot get it to work according to the user manual.
  - The equipment has been dropped and damaged.
  - The equipment has obvious signs of breakage.
5. Disconnect this equipment from any AC outlet before cleaning. Use a damp cloth. Do not use liquid or spray detergents for cleaning and keep this equipment away from humidity.

**Caution!** *To avoid short-circuiting and otherwise damaging the device, do not allow fluids to come in contact with the device. If fluids are accidentally spilled on the equipment, remove the affected unit from service as soon as possible and contact service personnel to verify that patient safety is not compromised.*

6. Put this equipment on a reliable surface during installation. Dropping it or letting it fall may cause damage. For plug-in equipment, the power outlet socket must be located near the equipment and must be easily accessible.

**Caution!** *To prevent overheating, do not cover the openings and place the device in direct sunlight or near radiant heaters.*

7. Make sure the voltage of the power source is correct before connecting the equipment to the power outlet. Position the power cord so that people cannot step on it. Do not place anything over the power cord. If the equipment is not used for a long time, disconnect it from the power source to avoid damage by transient over-voltage.

**Caution!** *Do not leave this equipment in an uncontrolled environment where the storage temperature is below 0°C (0°F) or above 50°C (122°F). this may damage the equipment.*

8. If your computer is losing dramatic time or the BIOS configuration is reset to the default, the battery has no power.

**Caution!** *Do not replace the battery yourself. Please contact a qualified technician or your retailer.*

*The computer is provided with a battery-powered, real-time clock circuit. There is a danger of explosion if the battery is incorrectly replaced. Replace only with BR2032 or an equivalent type recommended by the manufacturer.*

*Discard used batteries according to the manufacturer's instructions.*

**Caution!** *The battery charging indicator is not included with this device. It will be added to the finished system assembly and be shown with the completed system.*

9. Improper installation of VESA mounting can result in serious personal injury! VESA mount installation should be conducted by a professional technician. Please contact a service technician or your retailer if you need this service. The detailed operating procedure is specified in Chapter4.
10. CLASSIFICATION:
  - 1). Class I internal power
  - 2). No applied parts
  - 3). Continuous operation
  - 4). Not AP or APG categories

**Warning!** *This device is not suitable for use in the presence of a flammable anesthetic mixture with air, oxygen, nitrous oxide, or as part of a life-support system.*


11. Environmental protection: follow national requirements to dispose of the unit.
12. Maintenance: to properly maintain and clean the surfaces, use only approved products or clean with a dry applicator.

**Caution!** *When servicing the device, always use replacement parts that are qualified to Advantech standards. Advantech Medical cannot warrant or endorse the safe performance of third-party replacement parts for use with our medical device.*

13. Make sure the user does not contact SIP/SOPs and the patient at the same time.

14. When networking with electrical devices, the operator is responsible for ensuring that the resulting system meets the requirements set forth by the following standards:
  - EN 60601-1 (IEC 60601-1) Medical electrical equipment  
Part 1: General requirements for safety
  - EN 60601-1-2 (IEC 60601-1-2) Medical electrical equipment  
Part 1-2: General requirements for safety  
Collateral standard: Electromagnetic compatibility; Requirements and tests
15. Accessory equipment connected to analog and digital interfaces must be in compliance with the respective nationally harmonized IEC standards (i.e. IEC 60950 for data processing equipment, IEC 60065 for video equipment, IEC 61010-1 for laboratory equipment, and IEC 60601-1 for medical equipment.)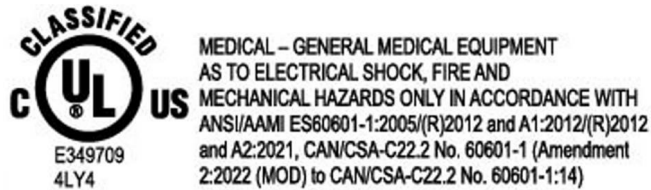

Furthermore, all configurations shall comply with the IEC 60601-1 system standard. Anyone who connects additional equipment to the signal input part or signal output part is configuring a medical system, and is therefore, responsible that the system complies with the requirements of the IEC 60601-1 system standard. The unit is for exclusive interconnection with IEC 60601-1 certified equipment in the patient environment and IEC 60XXX certified equipment outside of the patient environment. If in doubt, consult the technical services department or your local representative.

**Caution!** *Use a suitable mounting apparatus to avoid risk of injury. it shall be mounted by trained and authorized personnel with adequate allowances for quality of materials used to make appropriate connections.*

16. Grounding reliability can only be achieved when the equipment is connected to an equivalent receptacle marked "Hospital Only" or "Hospital Grade".
17. Use a power cord that matches the voltage of the power outlet, which has been approved and complies with the safety standard of your particular country.

**Note!** *Environmental protection*

*Follow national requirements to dispose of the unit.*

18. **WARNING** - Do not modify this equipment without authorization of the manufacturer.
19. **WARNING** - To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.
20. Remove the power cord to fully turn off the device.
21. Only use the power cord with the following specification: 18AWG min., type SJT, UL/CSA listed, length: 1.8m, hospital grade if for the USA/Canada market.
22. Installation is only to be carried out by manufacture authorized and trained personnel.
23. Device is intended to be used as SIP/SOP facing downward.

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## Explanation of Graphical Symbols

IEC 60878 and ISO 3864-B.3.6: Warning: dangerous voltage

ISO 7000-0434: Caution: consult ACCOMPANYING DOCUMENTS.


ISO 7000-1641: Follow operating instructions or consult the instructions for use.

IEC 60417 -5009: STAND-BY.

IEC 60417-5032: Alternating Current

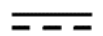IEC 60417-5031: Direct Current

IEC 60417-5021: Equipotentiality.

ISO 7010-M002: Follow the instructions for use.

## Disposing of your old product

### Within the European Union

EU-wide legislation, as implemented in each Member State, requires that waste electrical and electronic products carrying the mark (left) must be disposed of separately from normal household waste. This includes monitors and electrical accessories, such as signal cables or power cords. When you need to dispose of your display products, please follow the guidance of your local authority, or ask the shop where you purchased the product, or if applicable, follow any agreements made.

The mark on electrical and electronic products only applies to the current European Union Member States.

## FCC Class B

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a residential environment. This equipment generates, uses, and can radiate radio frequency energy. If not installed and used in accordance with this user manual, it may cause harmful interference to radio communications.

Note that even when this equipment is installed and used in accordance with this user manual, there is still no guarantee that interference will not occur. If this equipment is believed to be causing harmful interference to radio or television reception, this can be determined by turning the equipment on and off. If interference is occurring, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and the receiver.
- Connect the equipment to a power outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**Warning!** *Any changes or modifications made to the equipment which are not expressly approved by the relevant standards authority could void your authority to operate the equipment.*

**Warning!** *This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:*

- (1) this device may not cause harmful interference, and*
- (2) this device must accept any interference received, including interference that may cause undesired operation.*

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## List of Accessories

Before installing your Point-of-Care Terminal, ensure that the following materials have been received:

- POC-624-12 SERIES Point-of-Care Terminal
- Mounting kits and packet of screws.
- VESA mounting note x1
- China RoHs note x1

**Warning!** *There are no user-serviceable parts inside. Refer servicing to qualified personnel. Only the accessories indicated on the list of accessories above have been tested and approved to be used with the device. Accordingly, it is strongly recommended that only these accessories be used in conjunction with the specific device. Otherwise, the correct functioning of the device may be compromised.*

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## Additional Information and Assistance

Contact your distributor, sales representative, or Advantech's customer service center for technical support if you need additional assistance. Please have the following information ready before you call:

- Product name and serial number
- Description of your peripheral attachments
- Description of your software (operating system, version, application software, etc.)
- A complete description of the problem
- The exact wording of any error messages
- This equipment is a source of electromagnetic waves. Before using, please make sure that there are no EMI-sensitive devices in its surroundings which may malfunction.

### Environmental protection

- Follow national requirements to dispose of the unit.

**Manufacturer**

Advantech Co., Ltd.

No.1, Alley 20, Lane 26, Ruiguang Road Neihu District, Taipei,  
Taiwan 114, R.O.C.

TEL: (02) 2792-7818

**Distributed in Europe by:**

Advantech Europe Service Center

Science Park Eindhoven 5708

5692 ER Son en Breugel

The Netherlands

Phone: +31 (0)76 523 3100

Email: [iHealthcare.SalesSupport@advantech.nl](mailto:iHealthcare.SalesSupport@advantech.nl)

URL: <http://www.advantech.eu>

Visit the Advantech websites at [www.advantech.com](http://www.advantech.com) or [www.advantech.com.tw](http://www.advantech.com.tw) if  
you need more information.

# Contents

|                  |                                                                                     |           |
|------------------|-------------------------------------------------------------------------------------|-----------|
| <b>Chapter 1</b> | <b>General Information .....</b>                                                    | <b>1</b>  |
| 1.1              | Introduction .....                                                                  | 2         |
| 1.2              | Specifications .....                                                                | 2         |
|                  | Table 1.1: Specifications .....                                                     | 2         |
| 1.3              | Dimensions .....                                                                    | 4         |
|                  | Figure 1.1 Dimensions of the POC-624-12 Series Device .....                         | 4         |
|                  | Figure 1.2 Dimensions of the POC-624-12 Series Device with Smart Card Reader .....  | 4         |
|                  | Figure 1.3 Dimensions of the POC-624-12 Series Device with RFID & Camera .....      | 5         |
|                  | Figure 1.4 VESA Mounting of the POC-624-12 Series Device .....                      | 5         |
|                  | Figure 1.5 POC-624-12 Series Device Front Panel .....                               | 6         |
| 1.4              | Optional Modules .....                                                              | 6         |
|                  | 1.4.1 Cleaning and Disinfecting .....                                               | 6         |
| 1.5              | Operating Principle .....                                                           | 7         |
| 1.6              | Intended User Profile .....                                                         | 7         |
| <br>             |                                                                                     |           |
| <b>Chapter 2</b> | <b>System Setup .....</b>                                                           | <b>9</b>  |
| 2.1              | A Quick Tour of the POC-624-12 Series Device .....                                  | 10        |
|                  | 2.1.1 Front View .....                                                              | 10        |
|                  | Figure 2.1 Front View of the Point-of-Care Terminal .....                           | 10        |
|                  | 2.1.2 Rear View .....                                                               | 10        |
|                  | Figure 2.2 Rear View of the Point-of-Care Terminal .....                            | 10        |
|                  | Figure 2.3 Rear View of Multiple I/O ports (AC-In Model) .....                      | 11        |
| 2.2              | Installation Procedures .....                                                       | 11        |
|                  | 2.2.1 Connecting the Power Cord .....                                               | 11        |
|                  | Figure 2.4 Connecting the Power Cord .....                                          | 11        |
|                  | 2.2.2 Connecting the AC-In .....                                                    | 12        |
|                  | 2.2.3 Connecting the Ground Pin .....                                               | 12        |
|                  | Figure 2.5 POC-624-12 SERIES Equipotential Terminal Pin ....                        | 12        |
|                  | Figure 2.6 Grounding Cable with Connector .....                                     | 13        |
| 2.3              | Running the BIOS Setup Program .....                                                | 13        |
| 2.4              | Installing System Software .....                                                    | 13        |
| 2.5              | Installing the Drivers .....                                                        | 14        |
| 2.6              | Troubleshooting .....                                                               | 14        |
|                  | 2.6.1 POC not powering on .....                                                     | 14        |
|                  | 2.6.2 The POC system is powered on but does not boot into Windows ..                | 14        |
|                  | 2.6.3 AC power and all indicators show “On”, but the system does not power on ..... | 15        |
|                  | 2.6.4 No battery power .....                                                        | 15        |
| 2.7              | EMC Declaration .....                                                               | 16        |
| <br>             |                                                                                     |           |
| <b>Chapter 3</b> | <b>Operation and Safety .....</b>                                                   | <b>21</b> |
| 3.1              | General Safety Guide .....                                                          | 22        |
| 3.2              | Thermal .....                                                                       | 22        |
| 3.3              | Disconnect the Power .....                                                          | 23        |
| 3.4              | Proper Handling .....                                                               | 24        |
| 3.5              | Battery Warnings .....                                                              | 24        |
| 3.6              | Battery Safety Instructions .....                                                   | 25        |
| 3.7              | Emergency Scenarios .....                                                           | 29        |
| 3.8              | Battery Storage and Transportation .....                                            | 30        |

|                |           |                                                              |           |
|----------------|-----------|--------------------------------------------------------------|-----------|
|                | 3.9       | Battery Disposal.....                                        | 31        |
| <b>Chapter</b> | <b>4</b>  | <b>VESA Mount Installation .....</b>                         | <b>33</b> |
|                | 4.1       | VESA Mount Installation .....                                | 34        |
|                |           | Figure 4.1 VESA Mounting.....                                | 34        |
| <b>Chapter</b> | <b>5</b>  | <b>Driver Installation.....</b>                              | <b>35</b> |
|                | 5.1       | Driver Installation .....                                    | 36        |
| <b>Chapter</b> | <b>6</b>  | <b>PCM-8725 Connector Map.....</b>                           | <b>37</b> |
|                | 6.1       | The POC System Uses the PCM-8725 PCBA.....                   | 38        |
|                |           | Figure 6.1 Motherboard Top Side.....                         | 38        |
|                |           | Figure 6.2 Bottom Side.....                                  | 39        |
| <b>Chapter</b> | <b>7</b>  | <b>PCM-8725 Jumper Settings.....</b>                         | <b>41</b> |
|                | 7.1       | PCM-8725 Jumper Settings.....                                | 42        |
|                |           | Figure 7.1 Motherboard Top Side.....                         | 42        |
|                |           | Table 7.1: PCM-8725 Jumper Settings: .....                   | 42        |
|                |           | Table 7.2: CN6 ME Manufacturing Mode .....                   | 43        |
|                |           | Table 7.3: CN5 Clear CMOS.....                               | 43        |
|                |           | Table 7.4: CN8 System Reset (Reserved, Internal Test Only).. | 43        |
|                |           | Table 7.5: CN13 LVDS Voltage Setup .....                     | 43        |
|                |           | Table 7.6: CN40 Power Button (Reserved, Internal Test Only)  | 44        |
|                |           | Table 7.7: PCN4 Power Debug (Reserved, Internal Test Only)   | 44        |
|                |           | Table 7.8: SW1 Board Setup.....                              | 44        |
|                |           | Table 7.9: SW1 Pin 4 Speaker Function .....                  | 45        |
|                |           | Table 7.10: SW4 Panel Resolution/Type Setup .....            | 45        |
|                | 7.1.1     | POC-624-12 System Default .....                              | 45        |
| <b>Chapter</b> | <b>8</b>  | <b>Front Bezel Button .....</b>                              | <b>47</b> |
|                | 8.1       | Introduction .....                                           | 48        |
|                |           | Figure 8.1 Front button map .....                            | 48        |
| <b>Chapter</b> | <b>9</b>  | <b>Advanced BIOS Functions .....</b>                         | <b>49</b> |
|                | 9.1       | Advanced BIOS Function.....                                  | 50        |
| <b>Chapter</b> | <b>10</b> | <b>Temporarily Disable VMD for OS<br/>Image Restoration</b>  | <b>53</b> |
|                | 10.1      | Introduction .....                                           | 54        |
| <b>Chapter</b> | <b>11</b> | <b>Installing a PCIe Card .....</b>                          | <b>57</b> |
|                | 11.1      | Install PCIe Card.....                                       | 58        |
|                |           | 11.1.1 Installing the PCIe (x4) Card.....                    | 59        |
| <b>Chapter</b> | <b>12</b> | <b>Hot-Swap Battery Pack Operation...</b>                    | <b>63</b> |

|      |                                           |    |
|------|-------------------------------------------|----|
| 12.1 | Installing the Hot-Swap Battery Pack..... | 64 |
| 12.2 | Battery Pack Capacity Indicator.....      | 66 |
|      | Table 12.1: Capacity.....                 | 67 |
|      | Table 12.2: Capacity.....                 | 67 |
| 12.3 | Battery Pack Regular Use.....             | 68 |
| 12.4 | Battery Pack Storage .....                | 68 |
| 12.5 | Battery Operation Sequence.....           | 68 |

# Chapter 1

## General Information

## 1.1 Introduction

The POC-624-12 series device is a multimedia Intel® Core™ i7, i5, i3, or Celeron® processor device designed for mobile computing as a Point-of-Care terminal (POC). It is a PC-based system with a 23.8" wide screen TFT LCD display, HDMI, dual LAN ports. It has two COM ports, 4 x USB 3.2 Type-A, 1 x USB 3.2 Type-C, and a 24-bit stereo audio controller. With an optional 2.5" SSD, the POC-624-12 series device is a user-friendly computer. For system integrators, this highly integrated multimedia system lets you easily build a Point-of-Care terminal into your applications. The POC-624-12 series device makes it an ideal and safe point-of-care solution for patients and hospital practitioners. The POC-624-12 series device is specially designed to resist spills and water damage, and ensures dust resistance with its protected LCD and sealed ports. The POC-624-12 series device is a reliable solution for your application's processing requirements.

**Intended use** - The POC-624-12 series device is intended to serve as a Point-of-Care terminal (POC) for integration with hospital systems. The POC-624-12 series device is designed for general purpose medical computing in the hospital environment, for data collection, and for displaying information. It should not be used as a life-support system.

The latest version of this user manual is available for download from <http://support.advantech.com.tw/support/>

## 1.2 Specifications

**Table 1.1: Specifications**

|                  |                     |                                                                                                                                                                                               |
|------------------|---------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Computing System | CPU                 | Intel® Core™ i7-1185G7E (12M cache, up to 4.40 GHz)<br>Intel® Core™ i5-1145G7E (8M cache, up to 4.10 GHz)<br>For i3-1115G4E and Celeron® 6305E models, please contact Advantech for projects. |
|                  | Memory              | 8GB DDR4 (supports up to 64GB DDR4 SODIMM)                                                                                                                                                    |
|                  | Graphics Controller | Intel® Iris® Xe Graphics                                                                                                                                                                      |
|                  | Size                | 23.8-inch                                                                                                                                                                                     |
| Display          | Type                | 10-point multi touch P-CAP (clear glass); optional AG/AR support upon request                                                                                                                 |
|                  | Max. Resolution     | 1920 x 1080 (H x V)                                                                                                                                                                           |
|                  | Max. Colors         | 16.7 M colors (6-bits+A-FRC)                                                                                                                                                                  |
|                  | Pixel Pitch (µm)    | 247.5 x 247.5                                                                                                                                                                                 |
|                  | Viewing Angle       | 178/178°                                                                                                                                                                                      |
|                  | Luminance           | 250 cd/m2                                                                                                                                                                                     |
|                  | Backlight           | LED                                                                                                                                                                                           |
|                  | LCD MTBF            | 30,000 Hours                                                                                                                                                                                  |
|                  | Contrast Ratio      | 1000:1                                                                                                                                                                                        |
|                  | Mini PCIe           | 1 (full size)                                                                                                                                                                                 |
| Expansion Slots  | M.2 2230            | 1 (for Wi-Fi/Bluetooth module)                                                                                                                                                                |
|                  | PCIe x4**           | 1 (max card length: 167.65 mm / 6.6 in)<br>Either PCIe x4 or backup battery can be chosen, but not both.                                                                                      |

| Table 1.1: Specifications |                         |                                                                                                                                                                                                  |
|---------------------------|-------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| I/O Ports                 | USB 3.2 Gen 2 (Type-A)  | 4                                                                                                                                                                                                |
|                           | USB 3.2 Gen 2 (Type-C)  | 1 (with display function)                                                                                                                                                                        |
|                           | USB 2.0 Type-A          | 2                                                                                                                                                                                                |
|                           | HDMI                    | 1                                                                                                                                                                                                |
|                           | LAN isolation, 1.5KV    | 2                                                                                                                                                                                                |
|                           | Mic-In/Headphone-Out    | 1/1                                                                                                                                                                                              |
|                           | COM (RS232) 1.5KV       | 2                                                                                                                                                                                                |
| Power Input               | DC                      | 19V, max. 150W                                                                                                                                                                                   |
| Certifications            | EMC & Safety            | Medical CE, FCC (IEC 60601-1-2, 4th Edition)<br>EN 60601-1 Compliance<br>UL 60601-1                                                                                                              |
| Environment               | Temperature             | Operating: 0°C ~ +35°C,<br>Transport/Storage: 0°C ~ 50°C                                                                                                                                         |
|                           | Humidity                | Operating: 10% ~ 90%@40°C non-condensed,<br>Transport/Storage: 5% ~ 90%@40°C non-condensed                                                                                                       |
|                           | Shock Resistance        | 20G peak acceleration (11ms duration)                                                                                                                                                            |
|                           | Pressure                | 700-1013 hPa (Operation)<br>70-106 kPa (Storage)<br>70-106 kPa (Transportation)                                                                                                                  |
| Physical Characteristics  | Dimensions (W x D x H)  | 524 x 357 x 62 mm / 20.63 x 14.06 x 2.44 in                                                                                                                                                      |
|                           | Weight (bare system)    | 6.29 kg / 13.86 lb                                                                                                                                                                               |
|                           | VESA Mount              | 100 x 100, 75 x 75 mm                                                                                                                                                                            |
| Other Functions           | Internal Speaker        | 2x2 W                                                                                                                                                                                            |
|                           | Internal TPM            | 2.0 (default via fTPM -Firmware TPM)                                                                                                                                                             |
| Optional Configuration    | Operating System        | Windows 10 IoT Ent (64-bit only),<br>Linux (supported by request)                                                                                                                                |
|                           | Memory                  | Up to 64GB DDR4 SODIMM                                                                                                                                                                           |
|                           | Storage                 | NVME SSD 128G/ 256G/ 512GB PCIe interface or<br>128G / 256G / 512G / 1TB / 2TB 2.5" SATA SSD<br>available                                                                                        |
|                           | WLAN                    | Intel® Wi-Fi 6E AX210 or Intel Wireless-AC9260                                                                                                                                                   |
|                           | Bluetooth               | 5.3 / 5.1                                                                                                                                                                                        |
|                           | RFID                    | RFIDEAS KT-805N64KU<br>(The Pico Integrated Card Reader is a dual-band<br>card reader capable of reading both 125 kHz and<br>13.56 MHz credentials)<br>(choose either RFID or smart card reader) |
|                           | Smart Card Reader       | Smart Card (choose either RFID or smart card<br>reader)                                                                                                                                          |
|                           | Backup Battery          | One Internal Backup Battery, POC-IPSM90B(POC-<br>BAT-101-6A), Lithium-ion 8400mAh 10.8V 90.72Wh<br>3S3P). Either PCIe x4 or backup battery can be cho-<br>sen, but not both.                     |
|                           | Hot Swap Battery Module | Optional with 2 x batteries (choose either backup<br>battery or hot swap battery module), POC-<br>IPSM90B2 (POC-BAT-201-62)                                                                      |
|                           | Camera                  | 5MP, Auto Focus                                                                                                                                                                                  |
|                           | Touch Panel             | 23.8", clear glass, AR/AG (request by project)                                                                                                                                                   |

## 1.3 Dimensions

Dimensions: 523 x 356 x 62 mm (Unit: mm)



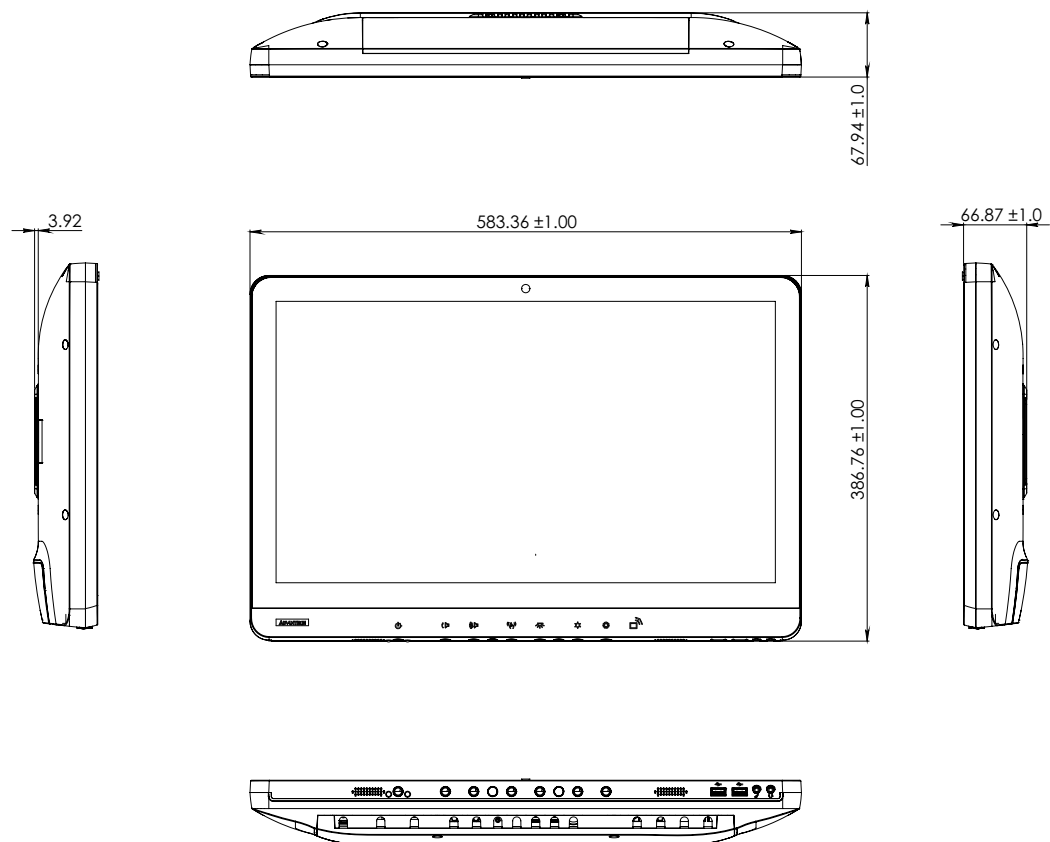
Figure 1.1 Dimensions of the POC-624-12 Series Device

Figure 1.2 Dimensions of the POC-624-12 Series Device with Smart Card Reader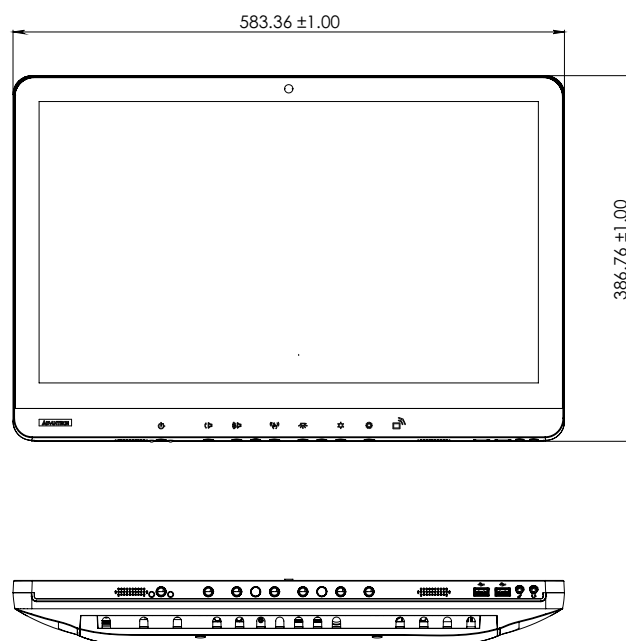

**Figure 1.3 Dimensions of the POC-624-12 Series Device with RFID & Camera**

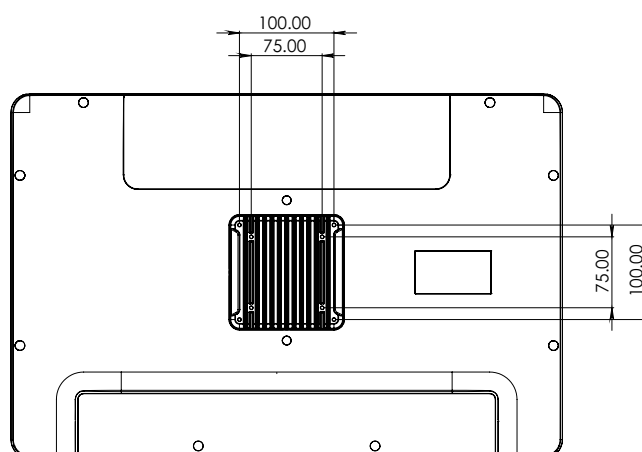**Figure 1.4 VESA Mounting of the POC-624-12 Series Device**

VESA Mounting: 75 x 75 mm; 100 x 100 mm


Please use M4x12L(mm) screw.

**Warning!** To prevent the risk of injury, utilize an appropriate mounting apparatus. Ensure that the mounting is performed by trained and authorized personnel, taking into account the use of high-quality materials for the installation.

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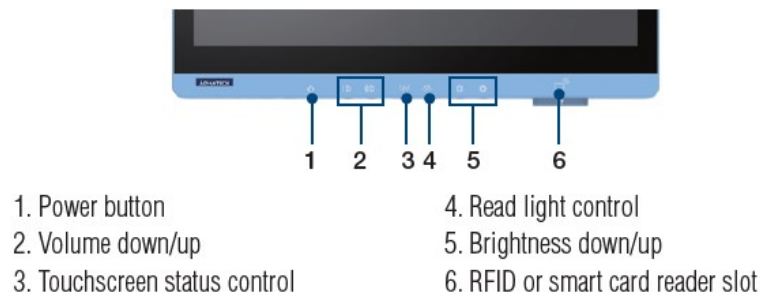**Figure 1.5 POC-624-12 Series Device Front Panel**

## 1.4 Optional Modules

- **Memory:** Up to 64GB DDR4 SODIMM
- **Primary Storage:** NVME (M2 2280, PCIe Interface)
- **Wifi and Bluetooth module:** Intel® Wi-Fi 6E AX210 and Intel® Wireless-AC9260
- **RFID module:** RFIDEAS KT-805N64KU
- Smart Card
- **Backup battery:** One Internal Backup Battery, POC-IPSM90B, Lithium-ion 8400mAh 10.8V 90.72Wh 3S3P.
- **Camera:** 5MP Auto Focus
- **Touchscreen:** Projected Capacitive / Optional AR/ Optional AG Coating

Above optional modules might mutually exclusive each other. Contact Advantech for more details.

### 1.4.1 Cleaning and Disinfecting

During normal use of the POC (Point-of-Care terminal), the device may become dirty and should be regularly cleaned.

#### Steps:

1. Prepare cleaning water.
2. Wipe the POC with a clean cloth that has been moistened in the cleaning solution.
3. Wipe thoroughly with a clean cloth.

#### Caution!

- *Do not immerse or rinse the POC or its peripherals. If you accidentally spill liquid on the device, disconnect the unit from the power source. Contact your IT support department regarding the continued safety of the unit before placing it back in operation.*
- *Do not spray cleaning agent on the chassis.*
- *Do not use disinfectants that contain phenol.*
- *Do not autoclave or clean the POC or its peripherals with strong aromatic, chlorinated, ketone, ether, or ether solvents, sharp tools, or abrasives. Never immerse electrical connectors in water or other liquids.*

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## 1.5 Operating Principle

The device provides input through the touch panel, hard keys located at the bottom of it, accessories through USB ports, or its LAN/WLAN connections. The device computes the input data with its processing unit and then outputs the generated data to the LCD panel, accessories, or other devices through its I/O ports or through its LAN/WLAN connections. The device is able to store data in its storage, and when the device is turned off, it maintains the data in the memory units of the storage.

## 1.6 Intended User Profile

### Intend user profile:

- Age: 18 to 65
- Weight: not relevant
- Health: not relevant
- Nationality: Global
- Patient state: patient will not be the operator.
- Part of the body or type of tissue applied to or interacted with: hands and fingers, expected contact time shall be less than 1 min.
- Education level: at least 8 years of intensive reading experience (school)
- Knowledge:
  - Minimum – read and understand “westernized Arabic” numerals when written in Arial font
  - can distinguish: every part of the body as described in the user manual
  - trained and authorized by manufacturer only
- To be considered as trained and authorized, a person must complete the training course of the manufacturer; see document number xxxxx for the qualification method. When considered necessary by the manufacturer, a technician shall be called back for retraining and annual training is also considered necessary.
- Language of understanding: English, and whenever other languages are required, a professional translation company shall translate and have the translation reviewed by the manufacturer. See SOP document number: XXXXX
- Experience: Mentally and physically competent, with specific medical training to understand basic knowledge for symbols.
- Permissible impairments:
  - Mild reading vision impairment or vision corrected to log MAR 0,2 (6/10 or 20/32)

- 
- One arm/hand system capable of guiding and holding the device
  - Average degree of aging-related short-term memory impairment
  - Hearing impaired by 40% resulting in 60% of normal hearing at 500 Hz to 2 kHz

# Chapter 2


## System Setup

## 2.1 A Quick Tour of the POC-624-12 Series Device

Before you start to set up the POC-624-12 series device, take a moment to become familiar with the locations and purposes of the controls, drives, connections and ports, which are illustrated in the figures below.

When you place the POC-624-12 series device upright on the desktop, its front panel appears as shown in Figure 2.1.

### 2.1.1 Front View

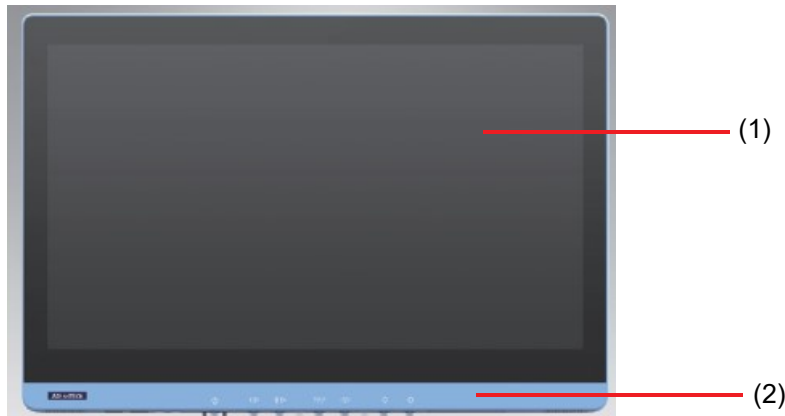
**Figure 2.1 Front View of the Point-of-Care Terminal**

#### Front Bezel view


- (1) LCD panel with touchscreen module
- (2) Power symbol w/ indicator light

### 2.1.2 Rear View

When you turn the Point-of-Care terminal around and look at its rear cover, the sunken I/O section is at the bottom of the panel PC, as shown in Figure 2.2, with a zoomed-in rear view in Figure 2.3. (The I/O section includes various I/O ports, including serial ports, HDMI port, Ethernet port, USB ports, and so on.)

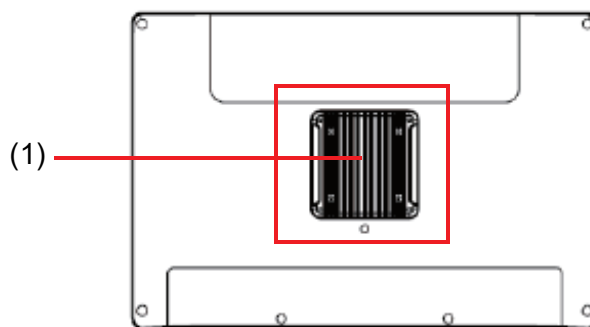**Figure 2.2 Rear View of the Point-of-Care Terminal**

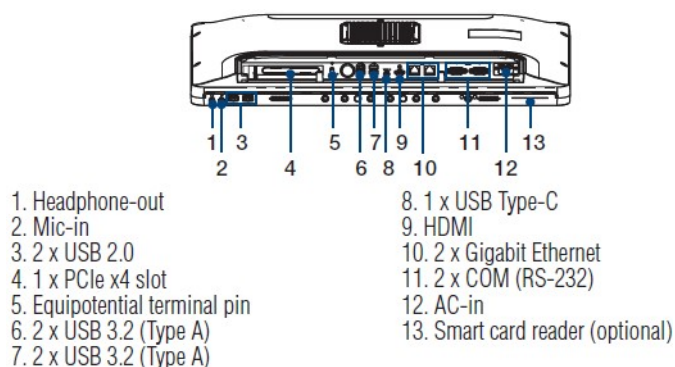**Figure 2.3 Rear View of Multiple I/O ports (AC-In Model)**

**Note!** The equipotential terminal needs a link to the hospital ground/earth system before system boot to protect the operator and system.

## 2.2 Installation Procedures

### 2.2.1 Connecting the Power Cord


The POC-624-12 series device can be powered by one of the below DC adapters.

| Manufacturer Model  | Voltage (Rating) | Current (Rating) | Note |
|---------------------|------------------|------------------|------|
| SINPRO HPU150A-107  | 19V              | 7.89A            |      |
| DELTA MDS-150AAS19F | 19V              | 7.89A            |      |

Be sure to always handle the power cords by holding the plug ends only.

Follow these procedures in order:

1. Connect the female end of the power cord to the AC Adapter.
2. Connect the 3-pin male plug of the power cord to an electrical outlet.

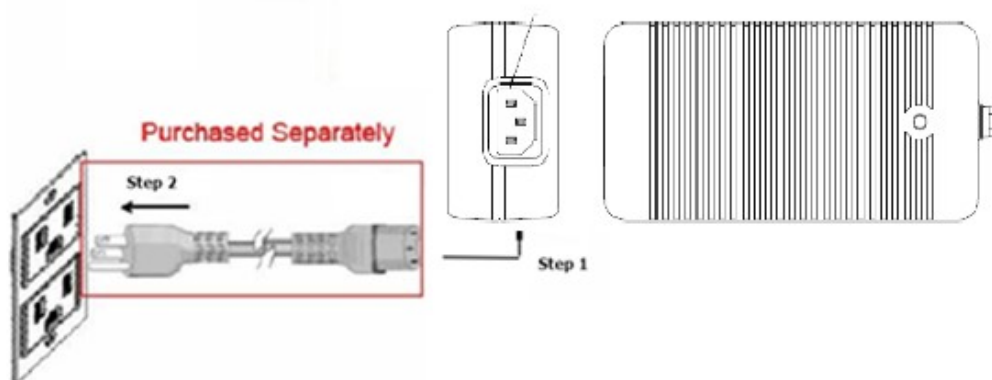**Figure 2.4 Connecting the Power Cord**

## 2.2.2 Connecting the AC-In

Use an AC adapter to plug into the AC-in on the POC system.

**Caution!** The AC adapter plug has a correct orientation. Please align the plug and connector correctly.

| Pin  | Function | Note |
|------|----------|------|
| 1, 2 | 19V      |      |
| 3, 4 | Return   |      |

**Warning!** Plugging in the power from the adapter in the wrong orientation may damage the POC system or adapter.

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
## 2.2.3 Connecting the Ground Pin

1. Find the equipotential terminal on the rear side of the POC terminal.  
An equipotential terminal is provided to optionally connect to a hospital ground/earth system.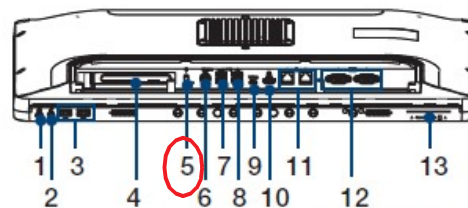

- |                                      |                                  |
|--------------------------------------|----------------------------------|
| 1. Headphone-out                     | 8. 2 x USB 3.2 (Type A)          |
| 2. Mic-in                            | 9. 1 x USB 3.2 (Type C)          |
| 3. 2 x USB 2.0                       | 10. HDMI                         |
| 4. 1 x PCIe x4 slot                  | 11. 2 x Gigabit Ethernet         |
| <u>5. Equipotential terminal pin</u> | 12. 2 x COM (RS-232)             |
| 6. DC-in                             | 13. Smart card reader (optional) |
| 7. 2 x USB 3.2 (Type A)              |                                  |

**Figure 2.5 POC-624-12 SERIES Equipotential Terminal Pin**

2. Prepare the grounding cable and the other terminal link to hospital ground/earth system.

Figure 2.6 Grounding Cable with Connector

3. Connect the cable with the connector to the equipotential terminal (See Figure 2.5).

## 2.3 Running the BIOS Setup Program

Your POC-624-12 series device was probably set up and configured by your dealer prior to delivery. You may still find it necessary to use the BIOS (Basic Input-Output System) setup program to change system configuration information, such as the current date and time or your type of hard drive. The setup program is stored in read-only memory. It can be accessed either when you turn on or reset the panel PC, by pressing the “F2” or “Del” key on your keyboard immediately after powering on the computer.

The settings you specify in the setup program are recorded in a special area of memory called CMOS RAM. This memory is backed up by a battery so that it will not be erased when you turn off or reset the system. Whenever you turn on the power, the system reads the settings stored in CMOS RAM and compares them to the equipment check conducted during the power on self-test (POST). If an error occurs, an error message will be displayed on the screen, and you will be prompted to run the setup program.

## 2.4 Installing System Software

Recent releases of operating systems from major vendors include setup programs which load automatically and guide you through hard disk preparation and operating system installation. The guidelines below will help you determine the steps necessary to install your operating system on the panel PC hard drive.

**Note!** *Some distributors and system integrators may have already pre-installed system software prior to shipment of your panel PC.*

If required, insert your operating system's installation or setup diskette into the external diskette drive until the release button pops out.

The BIOS supports system boot directly from the flash disk. You may also insert your system installation flash disk into a USB port.

Power on or reset the system by pressing the “Del” key to boot to the BIOS menu and adjust the boot device sequence.

You can also press the “F12” key when booting; a bootable device popup menu will appear. You can select the bootable device. The Point-of-Care terminal will automatically load the operating system from the flash disk.

If you are presented with the opening screen of a setup or installation program, follow the instructions on the screen. The setup program will guide you through preparation of your hard drive, and installation of the operating system.

## 2.5 Installing the Drivers

After installing your system software, you will be able to set up the chipset, graphics, Ethernet, and audio functions by driver.

The standard automatic installation procedures for installing the drivers are described in later chapters.

The various drivers and utilities on the CD-ROM disc have their own text files which help users install the drivers and understand their functions. These files are a very useful supplement to the information in this manual.

All drivers can be downloaded from the Advantech website.

## 2.6 Troubleshooting

When the system behaves abnormally, such as the conditions described below, please refer to the below instructions to troubleshoot.

### 2.6.1 POC not powering on

If the power button does not power on the unit (the Power ON green indicator is still off), please check:

1. Check the AC adapter cord and plug.
2. Check the AC adapter cord and plug are properly plugged into the wall socket.
3. If the AC adapter cord is not plugged in (battery power mode), please check that the battery is installed properly and that there is enough battery power.  
Please unplug the battery pack and press the power level indicator button on the battery pack.

The illuminated LED represents the battery level.

|                  | State-of-Charge      |
|------------------|----------------------|
| <b>LED Green</b> | <i>Current()</i> > 0 |
| LED1             | 0%~20%               |
| LED2             | 0~40%                |
| LED3             | 0~60%                |
| LED4             | 0~80%                |
| LED5             | 0~100%               |

### 2.6.2 The POC system is powered on but does not boot into Windows

The POC display shows a black background and some words.

1. Check that Windows OS is installed properly.

2. Check that the BIOS boot order is set up properly. Please set up the BIOS boot device with the below procedure:
  - a) press the “Del” key while booting up to get to the BIOS menu.
  - b) In Boot – Boot Option #1 – Select “Windows Boot Manager (Storage name)”.
  - c) In Save & Exit – Save Changes and Reset – Select “Yes”.
  - d) The POC system will reset and boot up.

### 2.6.3 AC power and all indicators show “On”, but the system does not power on

Check that the POC system power indicator is green and the LEDs are on.

Please contact Advantech customer service for further support.

### 2.6.4 No battery power

Please check:

1. Make sure the battery pack is installed properly. Please push the pack inside the battery slot.
2. Check that the DC-in connection is plugged in properly.
3. If a battery is used, the battery capacity should be under 98% to initiate the charge.

This design can avoid frequent charging.  
In other words, if the battery capacity is 99% or 100%, the charger will not charge the battery.

Contact your distributor, sales representative, or Advantech's customer service center for technical support if you need additional assistance. Please have the following information ready before you call:

- Product name and serial number
- Description of your peripheral attachments
- Description of your software (operating system, version, application software, etc.)
- A complete description of the problem
- The exact wording of any error messages
- Symptoms, photo, or video if available

## 2.7 EMC Declaration

| Guidance and Manufacturer's Declaration – Electromagnetic Emissions                                                                                                                                                         |                        |                                                                                                                                                                                                                                           |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The model POC-624-12 series device is intended for use in an electromagnetic environment as specified below. The customer or the user of the POC-624-12 series device should ensure that it is used in such an environment. |                        |                                                                                                                                                                                                                                           |
| Emissions test                                                                                                                                                                                                              | Compliance             | Electromagnetic environment – guidance                                                                                                                                                                                                    |
| RF emissions<br>CISPR 11                                                                                                                                                                                                    | Group 1                | The POC-624-12 series device uses RF energy only for its internal functions. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.                                        |
| RF emissions<br>CISPR 11                                                                                                                                                                                                    | Class B                | The POC-624-12 series device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |
| Harmonic emissions<br>IEC 61000-3-2                                                                                                                                                                                         | Class D                |                                                                                                                                                                                                                                           |
| Voltage fluctuations/<br>flicker emissions<br>IEC 61000-3-3                                                                                                                                                                 | Meets the requirements |                                                                                                                                                                                                                                           |

| Recommended Separation Distances Between Portable and Mobile Rf Communications Equipment and the POC-624-12 Series Device                                                                                                                                                                                                                                                                                                                                                                                                                                                                                      |                                                                        |                                         |                                          |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------|-----------------------------------------|------------------------------------------|
| The POC-624-12 series device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the POC-624-12 series device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the POC-624-12 series device as recommended below, according to the maximum output power of the communications equipment.                                                                                                                             |                                                                        |                                         |                                          |
| Rated maximum output power of the transmitter<br>W                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                             | Separation distance according to the frequency of the transmitter<br>m |                                         |                                          |
|                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                | 150 kHz to 80 MHz<br>$d = 1,2 \sqrt{P}$                                | 80 MHz to 800 MHz<br>$d = 1,2 \sqrt{P}$ | 800 MHz to 2,5 GHz<br>$d = 2,3 \sqrt{P}$ |
| 0,01                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                           | 0,12                                                                   | 0,12                                    | 0,23                                     |
| 0,1                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                            | 0,38                                                                   | 0,38                                    | 0,73                                     |
| 1                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                              | 1,2                                                                    | 1,2                                     | 2,3                                      |
| 10                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                             | 3,8                                                                    | 3,8                                     | 7,3                                      |
| 100                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                            | 12                                                                     | 12                                      | 23                                       |
| For transmitters rated at a maximum output power not listed above, the recommended separation distance $d$ in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where $P$ is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.<br>NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.<br>NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people. |                                                                        |                                         |                                          |

| <b>Guidance and Manufacturer's Declaration – Electromagnetic Immunity</b>                                                                                                                                           |                                                                                                                                                                                                                                                                                                                              |                                                                                                                                                                                                                                                                                                   |                                                                                                                                                                                                                                                                                                                                                                |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The POC-624-12 series device is intended for use in the electromagnetic environment specified below. The customer or the user of the POC-624-12 series device should ensure that it is used in such an environment. |                                                                                                                                                                                                                                                                                                                              |                                                                                                                                                                                                                                                                                                   |                                                                                                                                                                                                                                                                                                                                                                |
| <b>Immunity test</b>                                                                                                                                                                                                | <b>IEC 60601 test level</b>                                                                                                                                                                                                                                                                                                  | <b>Compliance level</b>                                                                                                                                                                                                                                                                           | <b>Electromagnetic environment –guidance</b>                                                                                                                                                                                                                                                                                                                   |
| Electrostatic discharge (ESD)<br>IEC 61000-4-2<br>Electrical fast transient/burst<br>IEC 61000-4-4                                                                                                                  | ±8 kV contact<br>±15 kV air                                                                                                                                                                                                                                                                                                  | ±8 kV contact<br>±15 kV air                                                                                                                                                                                                                                                                       | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.<br>Main power quality should be that of a typical commercial or hospital environment.                                                                                                                            |
| Surge<br>IEC 61000-4-5<br>Interruptions and voltage variations on power supply input lines<br>IEC 61000-4-11                                                                                                        | ±2 kV for power supply lines<br>±1 kV for input/output lines<br>±1 kV line(s) to line(s)<br>±2 kV line(s) to earth<br><br>Voltage Dips:<br>i) 0% reduction for 0.5 cycle at 50Hz,<br>0% reduction for 1 cycle at 50Hz<br>Performance Criterion B<br>ii) 70% reduction for 25/30 cycles at 50/60Hz<br>Performance Criterion C | ±2 kV for power supply lines<br>±1 kV for input/output lines<br>±1 kV line(s) to line(s)<br>±2 kV line(s) to earth<br><br>Voltage Dips:<br>i) 0% reduction for 0.5 cycle at 50Hz,<br>0% reduction for 1 cycle at 50Hz<br>Performance Criterion B<br>ii) 70% reduction for 25/30 cycles at 50/60Hz | If the user of the POC-624-12 series device requires continued operation during main power interruption. It is recommended that the POC-624-12 series device be powered from an uninterruptible power supply.<br><br>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. |
| Power frequency (50/60 Hz) magnetic field<br>IEC 61000-4-8                                                                                                                                                          | Voltage Interruptions:<br>0% reduction for 250/300 cycles at 50/60Hz<br>Performance Criterion C<br>30A/m                                                                                                                                                                                                                     | Performance Criterion C<br><br>Voltage Interruptions:<br>0% reduction for 250/300 cycles at 50/60Hz<br><br>Performance Criterion C<br>30A/m                                                                                                                                                       |                                                                                                                                                                                                                                                                                                                                                                |
| NOTE <i>UT</i> is the AC mains voltage prior to application of the test level.                                                                                                                                      |                                                                                                                                                                                                                                                                                                                              |                                                                                                                                                                                                                                                                                                   |                                                                                                                                                                                                                                                                                                                                                                |

| Guidance and Manufacturer's Declaration – Electromagnetic Immunity                                                                                                                                                                         |                                                                                      |                                                                      |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                         |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|----------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The POC-624-12 series device is intended for use in the electromagnetic environment specified below. The customer or the user of the POC-624-12 series device should ensure that it is used in such an environment.                        |                                                                                      |                                                                      |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                         |
| Immunity test                                                                                                                                                                                                                              | IEC 60601 test level                                                                 | Compliance level                                                     | Electromagnetic environment – guidance                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                  |
| Conducted RF<br><br>IEC 61000-4-6                                                                                                                                                                                                          | 3 and 6 Vrms<br><br>150 kHz to 80 MHz and specific ISM, AM frequency<br>3 and 6 Vrms | 3 and 6 Vrms<br><br>150 kHz to 80 MHz and specific ISM, AM frequency | Portable and mobile RF communications equipment should be used no closer to any part of the POC-624-12 series device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.<br><br><b>Recommended separation distance</b><br><br>$d = 1,2\sqrt{P}$<br><br>$d = 1,2\sqrt{P}$ 80 MHz to 800 MHz<br><br>$d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz<br><br>where $P$ is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and $d$ is the recommended separation distance in meters (m). |
| Radiated RF<br><br>IEC 61000-4-3                                                                                                                                                                                                           | 3V/m<br>80 MHz to 2.7 GHz, and specific frequency                                    | 3V/m<br>80 MHz to 2.7 GHz, and specific frequency                    | Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup><br><br>Interference may occur in the vicinity of equipment marked with the following symbol:<br><br>                                                                                                                                                                                                                                       |
| NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.<br>NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people. |                                                                                      |                                                                      |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                         |

<sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast, and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the POC-624-12 series device is used exceeds the applicable RF compliance level above, the POC-624-12 series device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the POC-624-12 series device.

<sup>b</sup> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

# Chapter 3

Operation and Safety

## 3.1 General Safety Guide

For your own safety and that of your equipment, always take the following precautions.

Disconnect the power plug (by pulling the plug, not the cord), from your computer if any of the following conditions exist:

- The power cord or plug is frayed or otherwise damaged.
- Any substance is spilled on the POC system.
- The POC system has been dropped or the case has been damaged.
- The POC system needs servicing or repair.
- The POC system needs cleaning.
- To remove/install any internal parts (excluding installing/removing a hotswap battery pack).

## 3.2 Thermal

The vent hole of the POC Series rear cover functions as a cooling airflow inlet and outlet. These air inlets and outlets transfer heat from inside the computer to the cooler air outside. Do not block these holes/vents with any soft material.

When using your POC series system, it is normal for the rear metal heatsink to get warm. The rear metal heatsink of the POC series system functions as a cooling surface that transfers heat from inside the computer to the cooler air outside.

Do not block this heatsink with any soft material.

To protect the battery pack from overheating, use/charge the battery in accordance with the instructions provided in the user manual.

- Warning!**
1. *Temperatures that are too high or too low may damage the battery.*
  2. *Do not place your POC system on a pillow or other soft material when it is on, as the material may block the airflow and cause the system to overheat.*

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Charge the battery and use it according to the datasheet.

- Warning!**
1. *If the temperature is too high or too low, your batteries may be damaged.*
  2. *If the temperature is too high or too low, your batteries may stop charging. A battery could restart charging again when its temperature returns to a normal temperature range.*

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### 3.3 Disconnect the Power

The only way to disconnect power completely is to unplug the power cord. Make sure at least one end of the power cord is within easy reach so that you can unplug the computer when you need to.

**Warning!** *Your AC cord came equipped with a three-wire grounding plug (a plug that has a third grounding pin). This plug will fit only a grounded AC outlet. If you are unable to insert the plug into an outlet because the outlet is not grounded, contact a licensed electrician to replace the outlet with a properly grounded outlet.*

*Do not defeat the purpose of the grounding plug.*

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**Warning!** *Never push objects of any kind into this product through the openings in the case.*

*Doing so may be dangerous and result in fire or harmful electric shock.*

*Never place anything on the system case before turning off the computer.*

*Never turn on your computer unless all of its internal and external parts are in place.*

*Operating the computer when it is open or missing parts can be dangerous and can damage your computer.*

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## 3.4 Proper Handling

Handle your POC system with care. The system is made of metal, glass, and plastic and contains sensitive electronic components. Because the POC system is a heavy object, please use two hands to handle it.

Hold the battery with both hands when installing, removing, or replacing a battery pack.

**Warning!** *Do not attempt to use the POC system if it is damaged (for example, the case is cracked or broken) as this may cause injury.*

**Caution!** 1) Set up and install the POC system on a stable work surface.

2) Do not push objects into the ventilation openings.

3) The SINPRO HPU150A-107 and DELTA MDS-150AAS19F adapters are medical grade adapters.

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## 3.5 Battery Warnings

Please read these safety instructions and warnings before using or charging the battery.

**Warning!** *Li-ion battery packs may explode and cause fire if defective or used incorrectly. To prevent this from happening, follow all usage instructions and safety guidelines provided in this user manual.*

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- Caution!**  1. Removal of the POC system battery pack and unplugging of the power cord will cause the connected POC system shut down, which may result in data losses. To prevent the loss of information, always save your work and shut down the POC system via the Windows OS before removing the battery or power cord.
2. Lithium batteries have a predetermined life span. Replace old batteries with new batteries when they have reached the end of their service life (the warranty is for 1 year or 300 cycles with 70% normal capacity, according to which occurs first).

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## 3.6 Battery Safety Instructions

1. The POC system should only be powered by an Advantech battery pack or compatible battery pack supplied by Advantech.

- Warning!**  1. Do not insert non-Advantech battery packs into the POC system.
2. Do not attempt to use battery packs of a different brand.

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2. Do not make the wrong polarity connection when charging and using battery packs. Always double check the battery installation direction before insertion or connection with the charger.

- Warning!**  Batteries installed in the wrong direction with the opposite polarity are at risk of exploding, which may cause fire.

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3. Only use the specific Li-ion battery charger provided by Advantech to charge the battery pack.

**Warning!** *Using a battery charger of a different brand may cause the battery to explode and result in fire.*

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4. Do not short the battery pack connector using metal or a wire lead.

**Warning!** *Wire lead shorts can cause the battery to explode and result in fire.*

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5. Do not drop the battery pack.

**Warning!** *If the battery cover is cracked, split, or broken, do not use the battery.*

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6. Do not expose the battery pack to fire or high temperatures.

**Warning!** *Exposure to high temperatures may cause the battery to explode.*

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7. Do not penetrate the battery with nails, strike the battery with a hammer, step on the battery, or otherwise subject it to significant impact or shock.

**Warning!** *Batteries are at risk of exploding and may cause fire if subjected to significant impact.*

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8. Do not expose the battery pack to moisture or water.
9. Do not disassemble or modify the battery pack. The battery pack contains safety and protection circuits that if damaged, may cause the battery to overheat, explode, or ignite. There are no user-serviceable parts inside the battery pack.
10. Always refer to the user manual for charging instructions.
11. Charge the battery pack in indoor environments with a controlled temperature.

**Caution!** *Exposure to extreme temperatures can damage the battery and reduce the battery life.*

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12. Do not leave fully charged battery packs connected to the computer for prolonged periods of time.

**Caution!** *When the battery is fully recharged, disconnect it from the charger or unplug the charger power cable.*

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13. Do not discharge the battery pack using any device other than the POC system or a device specified by Advantech.

**Warning!** *Using the battery pack to power devices other than the POC system may damage the battery, limit performance, or reduce the battery life. Additionally, exposure to abnormal current may cause the battery pack to overheat, explode, or ignite, causing serious injury.*

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14. Use/discharge the battery pack in indoor environments with a controlled temperature.

**Caution!** *Using the battery in environments with temperatures outside the acceptable range may damage the battery, limit performance, or reduce the battery life.*

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15. Inspect all battery packs before use.

**Warning!** 1) *Never use battery packs that are visibly damaged or may have internal damage.*  
2) *Never remove the battery's outer cover or use a battery pack with a damaged cover.*

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16. Replace the entire battery pack with a new battery pack when the time between charges decreases significantly.

**Caution!** *When the time between charges decreases significantly, check the voltage of the battery pack before charging. If the battery performance drops significantly, please do not use this battery pack.*

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## 3.7 Emergency Scenarios

If you notice any of the following, stop charging or using the battery immediately:

1. The battery has become swollen, bulging, or deformed.
2. The battery is leaking fluid, smoke, or a foul odor.
3. The battery temperature is extremely hot.
4. The battery appears abnormal in any way.
5. The battery cover is cracked, split, or broken.
6. The battery has been exposed to moisture.

**Warning!** 1. *If any of the above conditions are observed, place the battery pack and charger outside on a concrete floor, away from any flammable materials for approximately 15 minutes and contact Advantech. Do not attempt to reuse the battery pack.*

2. *In the event that the battery is leaking and the fluid gets into your eyes, rinse well with water and immediately seek medical care. If left untreated, exposure to battery fluid could damage your eyes.*

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## 3.8 Battery Storage and Transportation

1. Store the battery in a dry environment with a room temperature of 0 ~ 25°C (32 ~ 77°F) for optimum health.
2. Do not expose the battery pack to direct sunlight (heat) or store the battery pack inside vehicles in hot weather for extended periods.
3. When transporting or temporarily storing the battery in a vehicle, the internal vehicle temperature should be greater than -20°C (-4°F) but no more than 50°C (122°F).
4. If possible, remove the battery pack from the device if the device will be unused for several months.
5. Do not remove individual battery packs from the original packaging until required for use.

**Warning!** *To reduce the risk of damage, do not expose battery packs to high temperatures for extended periods of time. This may damage the battery and cause fire.*

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**Caution!**

1. *Stored batteries should be checked every 3 months. If the capacity is less than 50%, recharge the battery before returning it to storage.*
2. *Batteries may leak if left in the device for extended periods of time. After long periods of storage, check the battery pack before charging.*

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## 3.9 Battery Disposal

When the battery pack has reached the end of its service life, do not use general household waste for disposal. Follow local laws and government regulations for correct battery disposal.

**Caution!** *Cover terminals with tape to prevent inadvertent contact with other batteries or metal objects.*

**Warning!** 1. *To reduce the risk of fire or burns, do not disassemble, crush, puncture, expose to fire or water, or short the battery's external contacts.*

2. *Used batteries may still have a partial charge. If partially charged batteries come into contact with other batteries or metal objects, the remaining stored energy may be discharged and cause a fire or explosion.*

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## VESA Mount Installation

## 4.1 VESA Mount Installation

The POC Series also provides standard VESA mounting to help system integrators conveniently integrate the panel PC into their system.

Never use mounting brackets except as provided by Advantech to prevent unreliable mounting of the POC series device. VESA mount installation should be carried out by a professional technician; please contact a service technician or your retailer if you need this service.

Installation instructions:

1. First attach the wall-mount to the heatsink of the POC-624-12 series device, securing it in place with four of the philips-head screws provided.
2. Mount it to the wall, stand, or other flat surface.

**Warning!** *Be sure to secure the screws of the mounting bracket tightly. A loose joint between the POC series device and mounting bracket may create danger of injury.*

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## Driver Installation

## 5.1 Driver Installation

The POC system supports Windows (WIN10 IOT/RS5 version or later, 64-bit only). It no longer supports 32-bit drivers.

**Warning!** Please use a clean OS installation to install the drivers; otherwise, it may cause unexpected errors.

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### Windows Driver List:

Please follow your OS version to install the proper drivers.

Please follow the below sequence to install drivers.

| Install sequence | Folder Name                     | Note                                                                                                                                                                                                                             |
|------------------|---------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 1                | Chipset                         | Please install the chipset driver first.                                                                                                                                                                                         |
| 2                | Graphics                        | Because Windows update will downgrade this graphics driver, please install the graphics driver again after a Windows update. After the 2nd graphics driver installation, Windows update will not downgrade the graphics driver.  |
| 3                | Audio                           |                                                                                                                                                                                                                                  |
| 4                | LAN                             | 1. Install the LAN driver first.<br>2. Install the Proset utility tool after the above LAN driver is installed.                                                                                                                  |
| 5                | AMT/ME                          | Intel® Management Engine Driver                                                                                                                                                                                                  |
| 6                | RST/VMD                         | Intel® RST/VMD Driver. For a different Win10 version, you need to install the specific RST/VMD driver.<br>1. For WIN10 20H2 and later/WIN11, install RST version 18.7<br>2. For WIN10 1909 and earlier, install RST version 18.4 |
| 7                | Serial I/O                      |                                                                                                                                                                                                                                  |
| 7                | Wireless Card (Wi-Fi+Bluetooth) | Option<br>Please install both Wi-Fi and Bluetooth drivers.                                                                                                                                                                       |
| 8                | RFID                            | Optional                                                                                                                                                                                                                         |
| 9                | Smart Card                      | Optional                                                                                                                                                                                                                         |

# Chapter 6

## PCM-8725 Connector Map

## 6.1 The POC System Uses the PCM-8725 PCBA

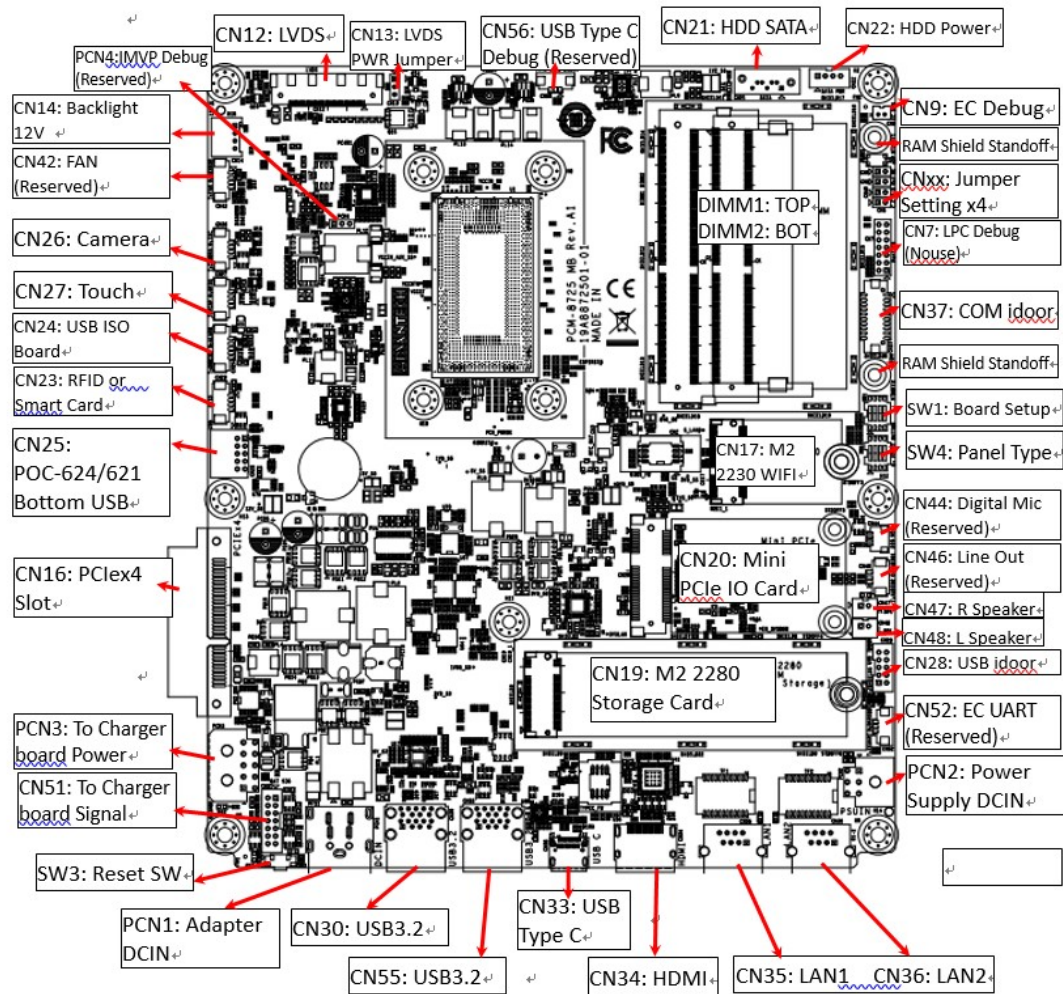
Figure 6.1 Motherboard Top Side

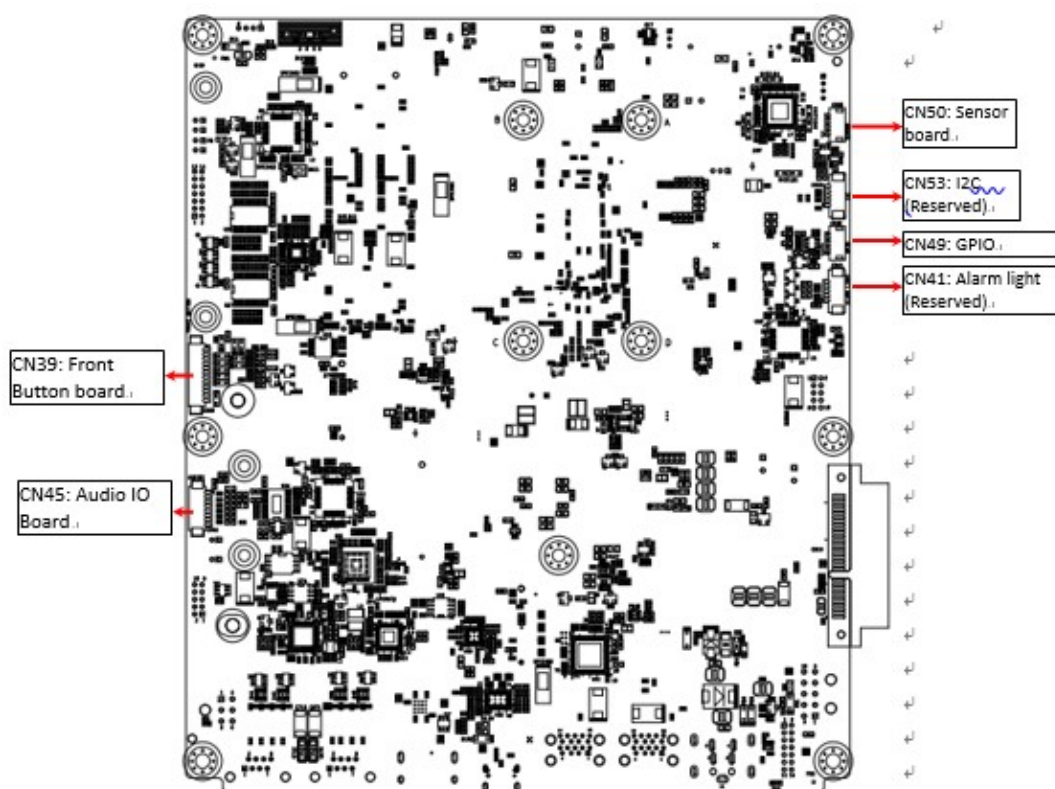
Figure 6.2 Bottom Side


# Chapter 7

## PCM-8725 Jumper Settings

## 7.1 PCM-8725 Jumper Settings

The POC system uses the PCM-8725 PCBA. All Jumpers/DIP switches are located on the top side of the motherboard.

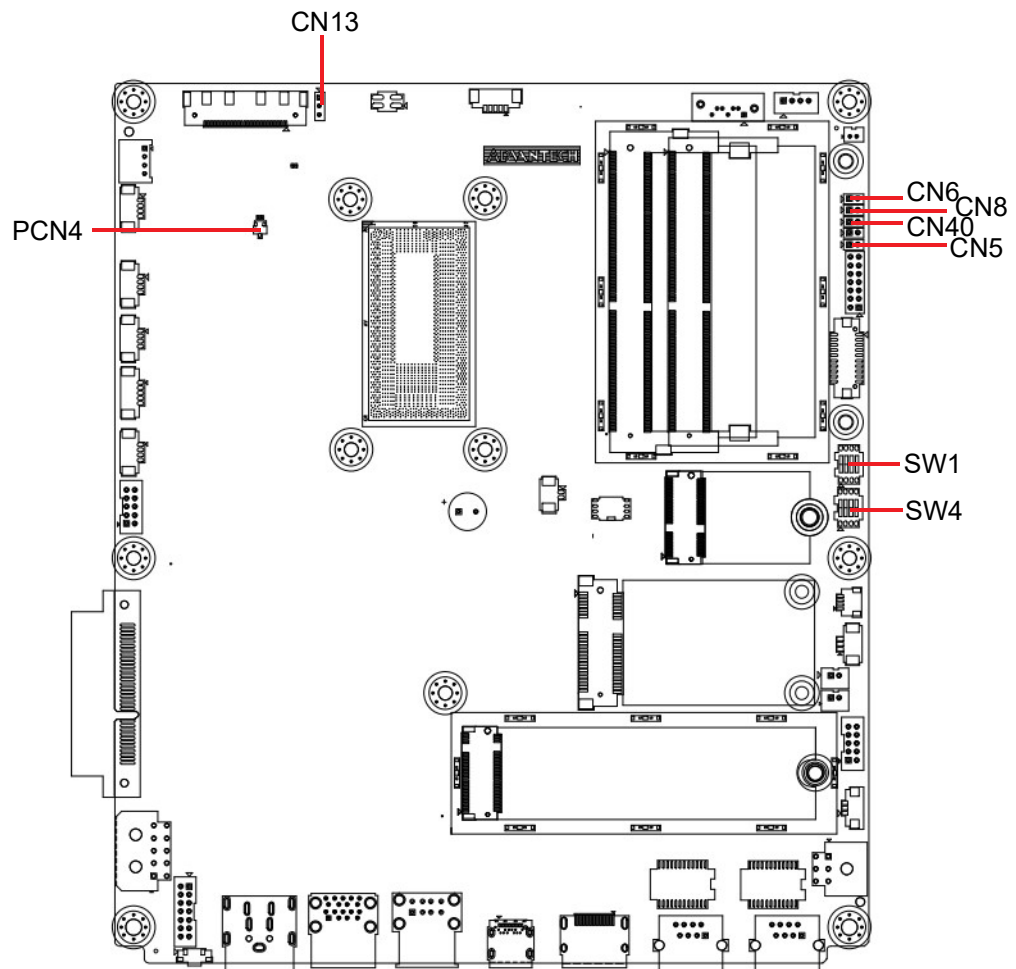**Figure 7.1 Motherboard Top Side**

**Table 7.1: PCM-8725 Jumper Settings:**

|      |                                             |
|------|---------------------------------------------|
| CN6  | ME Manufacturing Mode                       |
| CN5  | Clear CMOS                                  |
| CN8  | System Reset (Reserved, Internal Test Only) |
| CN13 | LVDS Voltage                                |
| CN40 | Power Button (Reserved, Internal Test Only) |
| PCN4 | Power Debug (Reserved, Internal Test Only)  |
| SW1  | Board Setup                                 |
| SW4  | Panel Setup                                 |

**Table 7.2: CN6 ME Manufacturing Mode**

|                 |                            |
|-----------------|----------------------------|
| Description     | ME Manufacturing Mode      |
| Setting         | Function                   |
| (1-2)           | ME Manufacturing Mode      |
| (No Connection) | Normal Operation (Default) |

**Table 7.3: CN5 Clear CMOS**


|                 |                            |
|-----------------|----------------------------|
| Description     | Clear CMOS Setup           |
| Setting         | Function                   |
| (1-2)           | Clear CMOS Setup           |
| (No Connection) | Normal Operation (Default) |

**Table 7.4: CN8 System Reset (Reserved, Internal Test Only)**

|                 |                            |
|-----------------|----------------------------|
| Description     | Reset system button        |
| Setting         | Function                   |
| (1-2)           | System Reset               |
| (No Connection) | Normal Operation (Default) |


**Table 7.5: CN13 LVDS Voltage Setup**

|             |                                   |
|-------------|-----------------------------------|
| Description | Select panel LVDS voltage setting |
| Setting     | Function                          |
| (1-2)       | Panel LVDS voltage 5V (Default)   |
| (2-3)       | Panel LVDS voltage 3.3V           |

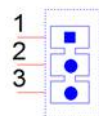
**Table 7.6: CN40 Power Button (Reserved, Internal Test Only)**

|                 |                                |
|-----------------|--------------------------------|
| Description     | Power Button Signal            |
| Setting         | Function                       |
| (Pin 2)         | Short pin 1 to Power ON system |
| (No Connection) | Normal Operation (Default)     |

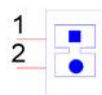
**Table 7.7: PCN4 Power Debug (Reserved, Internal Test Only)**

|                 |                                 |
|-----------------|---------------------------------|
| Description     | Power Debug, Internal Test Only |
| Setting         | Function                        |
| (No Connection) | Normal Operation (Default)      |

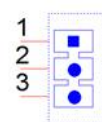**Table 7.8: SW1 Board Setup**

|             |                         |
|-------------|-------------------------|
| Description | Select Board setting    |
| Setting     | Function                |
| (SW1-1/2/3) | Select board type       |
| (SW1-4)     | Force Speaker always ON |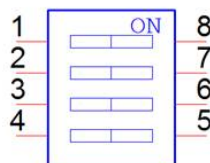


| SW1 Pin 1 | SW1 Pin 2 | SW1 Pin 3 | Board Configuration      |
|-----------|-----------|-----------|--------------------------|
| Hi(OFF)   | Hi(OFF)   | Hi(OFF)   | Board Config 1 (Default) |
| Hi(OFF)   | Hi(OFF)   | Low(ON)   | Board Config 2           |
| Hi(OFF)   | Low(ON)   | Hi(OFF)   | Board Config 3           |
| Hi(OFF)   | Hi(OFF)   | Hi(OFF)   | Board Config 4           |
| Low(ON)   | Hi(OFF)   | Hi(OFF)   | Board Config 5           |
| Low(ON)   | Hi(OFF)   | Low(ON)   | Board Config 6           |
| Low(ON)   | Low(ON)   | Hi(OFF)   | Board Config 7           |
| Low(ON)   | Hi(OFF)   | Hi(OFF)   | Board Config 8           |

**Table 7.9: SW1 Pin 4 Speaker Function**

|         |                                                                   |
|---------|-------------------------------------------------------------------|
| Hi(OFF) | Speaker Mute when audio plug in – with Audio board system default |
| Low(ON) | Speaker Always ON –without Audio board system default             |

**Table 7.10: SW4 Panel Resolution/Type Setup**

|             |                      |
|-------------|----------------------|
| Description | Select panel setting |
| Setting     | Function             |
| (SW2-1~4)   | Select panel type    |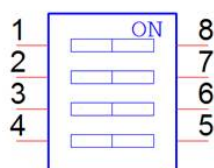


| SW4 Pin 1 | SW4 Pin 2 | SW4 Pin 3 | SW4 Pin 4 | Panel Resolution    | LVDS Channel   |
|-----------|-----------|-----------|-----------|---------------------|----------------|
| Hi(OFF)   | Hi(OFF)   | Hi(OFF)   | Hi(OFF)   | 1920x1200           | 24 bits Dual   |
| Low(ON)   | Hi(OFF)   | Hi(OFF)   | Hi(OFF)   | 1920x1080 (Default) | 24 bits Dual   |
| Hi(OFF)   | Hi(OFF)   | Low(ON)   | Hi(OFF)   | 1600x900            | 24 bits Dual   |
| Low(ON)   | Low(ON)   | Low(ON)   | Hi(OFF)   | 1366x768            | 24 bits Single |
| Low(ON)   | Hi(OFF)   | Hi(OFF)   | Low(ON)   | 1280x1024           | 24 bits Dual   |
| Low(ON)   | Hi(OFF)   | Low(ON)   | Low(ON)   | 1024x768            | 24 bits Single |
| Low(ON)   | Low(ON)   | Low(ON)   | Low(ON)   | 800x600             | 18 bits Single |

### 7.1.1 POC-624-12 System Default

POC-624-12, with audio board, 5V LVDS Panel

| SW1 Pin 1 | SW1 Pin 2 | SW1 Pin 3 | SW1 Pin 4 | SW2 Pin 1 | SW2 Pin 2 | SW2 Pin 3 | SW2 Pin 4 | CN13  |
|-----------|-----------|-----------|-----------|-----------|-----------|-----------|-----------|-------|
| Hi(OFF)   | Hi(OFF)   | Hi(OFF)   | Hi(OFF)   | Low(ON)   | Hi(OFF)   | Hi(OFF)   | Hi(OFF)   | (1-2) |

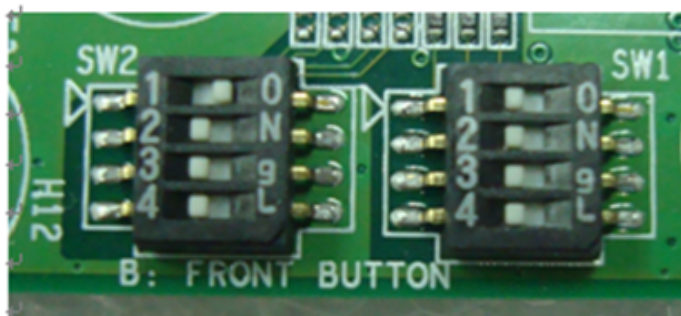# Chapter 8

## Front Bezel Button

## 8.1 Introduction

This section gives information on the POC system front buttons.

Figure 8.1 Front button map

### Button descriptions:

|                     |                                                                                    |                                                                                                                                                                                     |
|---------------------|------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Power button        |   | Press this button to power the system on/off. When the system is ON, this icon will become green. The LED will be off when the system is off.                                       |
| Reduce volume       |   | Press this button to reduce speaker and headphone volume.                                                                                                                           |
| Increase volume     |   | Press this button to increase speaker and headphone volume.                                                                                                                         |
| Touch disable       |   | Press this button to disable/enable the touch function. When the touch function is enabled, this icon will become green. When the touch function is disabled, this LED will be off. |
| Read light          |   | Press this button to enable/disable the read light.                                                                                                                                 |
| Reduce brightness   |   | Press this button to reduce the LCD backlight brightness.                                                                                                                           |
| Increase brightness |  | Press this button to increase the LCD backlight brightness.                                                                                                                         |

### Button combinations:

A special function is the “backlight off” function. When you press both the “reduce brightness” and “increase brightness” buttons for half a second, the LCD backlight will be turned off. If you press both buttons for half a second again, the LCD backlight will be turned on.

When the backlight is off, the POC system is still running, and it will not impact any program operations.

This “backlight off” function can be used in hospital environment.

If the operator needs to check a photo in a dark environment, the operator can turn off the backlight quickly to prevent interference from the dark environment.

The operator can turn off the backlight at night to prevent interference with a patient's sleep.

This “backlight off” event will turn the backlight back on automatically after the next system shutdown and system on cycle.

# Chapter 9

## Advanced BIOS Functions

---

## 9.1 Advanced BIOS Function

This is an introduction to the advanced functions in the POC system BIOS menu.

### 1. Power Button Function Enable/Disable

You can enable/disable the power button function in the BIOS menu.

If you disable the power button in S0 (System ON) status, the power button will not work. This means the user will not be able to turn off the system with the power button. The user needs to use software to turn off the system. This function can prevent shutting down the system by accidentally touching the power button.

#### BIOS Menu location:

BIOS Menu – Advanced – IT5121 HW Monitor – Power Button Function

- Enable: Power button function work when system in S0 (System ON) status. (Default)
- Disable: The power button function not work when the system is in S0 (System ON) status.  
Disabling the power button function will cause the system to not be powered on by pressing the power button. When you set this function to “disable”, BIOS will set the “State After G3” to “Power On” automatically. Then you can plug in a power adapter to turn on the POC system.

### 2. Brightness Button Control

You can enable/disable the LCD backlight button function in the BIOS menu.

If you disable the LCD backlight button, this button will not work. The user will not be able to adjust LCD backlight luminance with this button. This function can prevent accidental adjustment of the LCD backlight luminance.

#### BIOS Menu location:

BIOS Menu – Advanced – IT5121 HW Monitor – Brightness Button Control

- Enable: Front bezel LCD backlight adjust button control function is enabled. (Default)
- Disable: Front bezel LCD backlight adjust button control function is disabled.

### 3. Volume Button Control

You can enable/disable the front volume button function in the BIOS menu.

If you disable the front volume button, this button will not work. Then the user will not be able to adjust the speaker volume with this button. This function can prevent accidentally adjusting the speaker volume.

#### BIOS Menu location:

BIOS Menu – Advanced – IT5121 HW Monitor – Volume Button Control

- Enable: Front bezel volume adjust button control function is enabled. (Default)
- Disable: Front bezel volume adjust button control function is disabled.

#### 4. Touch Button Control

You can enable/disable the touch button function in the BIOS menu.

If you disable the touch button, this button will not work. Then the user will not be able to turn off touch with this button. The touch function will always be on.

##### BIOS Menu location:

BIOS Menu – Advanced – IT5121 HW Monitor – Touch Button Control

- Enable: Front bezel Touch button control function is enabled. (Default)
- Disable: Front bezel Touch button control function is disabled.

#### 5. Read light Button Control

You can enable/disable the read light button function in the BIOS menu.

If you disable the read light button, this button will not work. The user will not be able to turn on the read light with this button. The read light function will always be off.

##### BIOS Menu location:

BIOS Menu – Advanced – IT5121 HW Monitor – Read light Control

- Enable: Front bezel read light button control function is enabled. (Default)
- Disable: Front bezel read light button control function is disabled.

#### 6. EC Beep Function

You can enable/disable the EC beep function in the BIOS menu.

If you disable the EC beep function, EC will not generate a beep sound when you press the front bezel button.

##### BIOS Menu location:

BIOS Menu – Advanced – IT5121 HW Monitor – Beep Sound Function

- Enable: A beep sound will be audible when the front bezel button is pressed. (Default)
- Disable: There will be no beep sound when the front bezel button is pressed.

#### 7. Setup Popup Menu F12

You can press the “F12” key for the setup popup menu in system boot.

When you press the “F12” key during bootup, the BIOS will display a bootable device menu and you can select the appropriate boot device.

# Chapter 10


Temporarily Disable  
VMD for OS Image  
Restoration

## 10.1 Introduction


The POC system supports the latest generation of Intel® VMD Disk Controller. However, some old OS and tools cannot recognize this new controller. It may not detect the internal NVME storage when you restore from the OS image.

The following description shows step-by-step how to disable VMD in BIOS.


1. Boot to the BIOS setup menu. (Press the Del or Esc key during system boot.)
2. In the BIOS menu - Chipset - System Agent (SA) Configuration - VMD setup menu.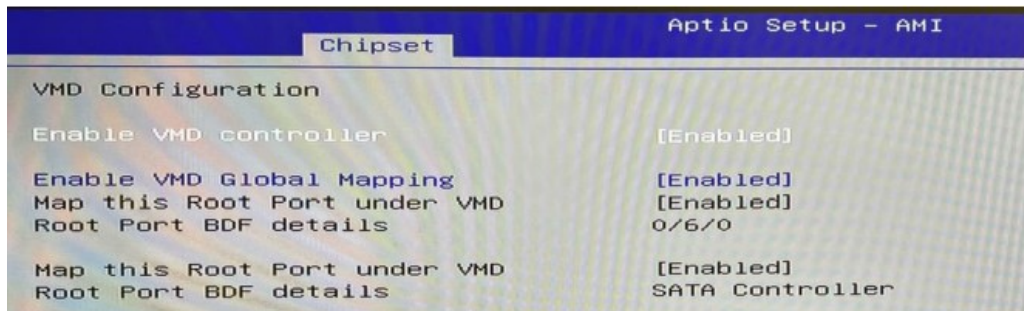

3. Change Enable VMD controller to Disabled.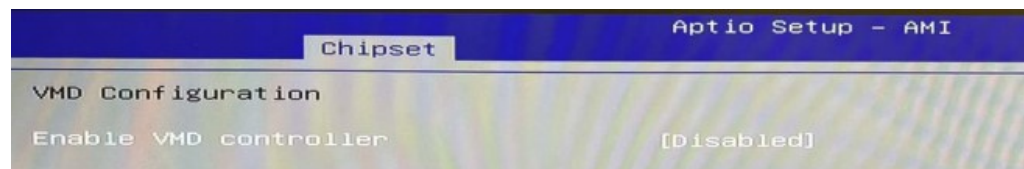

4. Save the settings and restart.
5. You can use OS restore tools, such as the GHOST tool to restore the OS image to the POC system.
6. After OS restoration is complete, please enable the VMD controller before OS boot.  
Boot to the BIOS setup menu. (Press the Del or Esc key during system boot.)
7. Go to the BIOS menu - Chipset - System Agent (SA) Configuration - VMD setup menu.
8. Set all four items to Enabled.

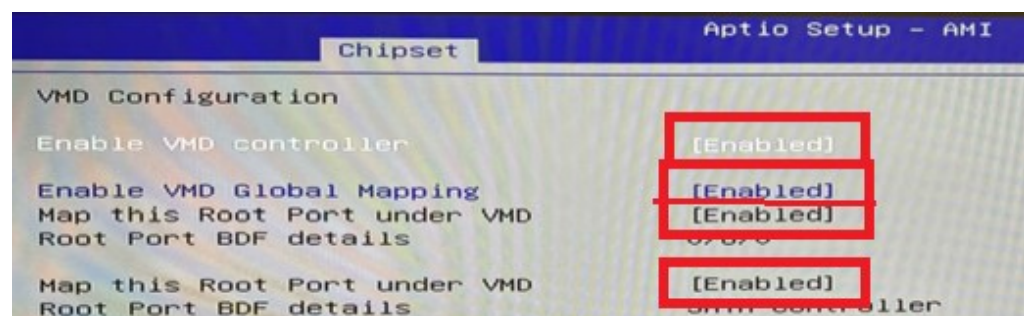If the last two "Map this Root Port under VMD" items cannot be selected, you can set "Enable VMD Global Mapping" to Disabled. The last two items will then be selectable.

Please set the above all four items to Enabled.

9. Save the settings and restart.

10. System boot into your OS image properly.

**Caution!** *Without enabling the VMD controller in BIOS, it may cause boot up of the OS to fail.*

# Chapter 11

## Installing a PCIe Card

## 11.1 Install PCIe Card

The POC system contains one PCIe x4 slot for expansion. The following section describes the relevant PCIe card installation information.

### 1. PCIe Card Dimensions


This slot supports the standard-height 111.15mm (4.376 inches) card. Note that card height is measured from the bottom of the edge finger to the top of the card.

The card length limit is 168mm (6.6 inches).

The length over this limit might impact the reserved 2.5" SSD space, causing it to not be installed properly.

This PCIe x4 Card and internal backup battery module are mutually exclusive. You cannot install both modules in the same location.

**Caution!** Install a PCIe card with length shorter than 168mm.

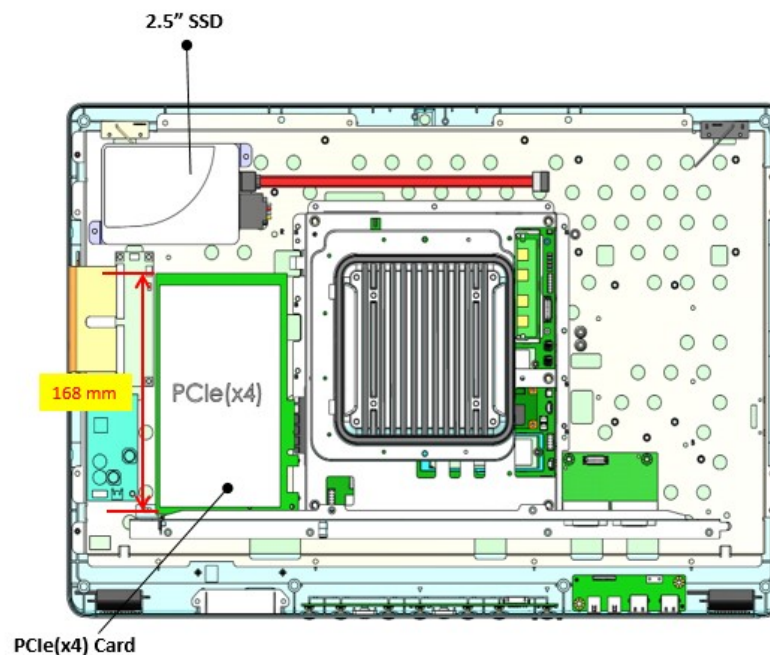### 2. PCIe Card Power

Because system cannot dissipate heat through enclosure vent holes, adding a high power/thermal PCIe card might lead to overheating conditions.

Advantech recommends that the PCIe Card thermal dissipation wattage should not be over 5W.

The customer needs to consider the total thermal solution and if the card's thermal dissipation in excess of the above criteria. It may be necessary to add a fan or control the temperature of the environment through other means.

**Warning** *Installing a high-power PCIe card might cause the system to overheat. This may cause the system to shut down or restart.*

### 3. PCIe Card Installation


The following section describes how to install the PCIe card.

**Warning** *Because of the danger of electric shock, never remove the cover of a device while it is in operation or connected to a power outlet.*


#### 11.1.1 Installing the PCIe (x4) Card


1. Remove the fixed bracket screw.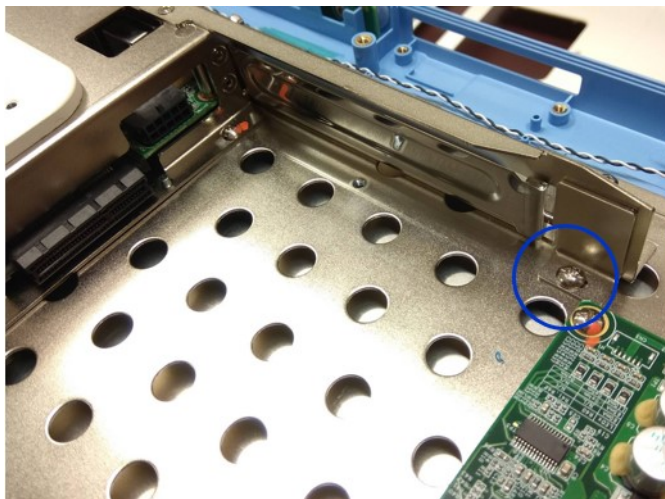


2. Remove the fixed bracket and dummy plate.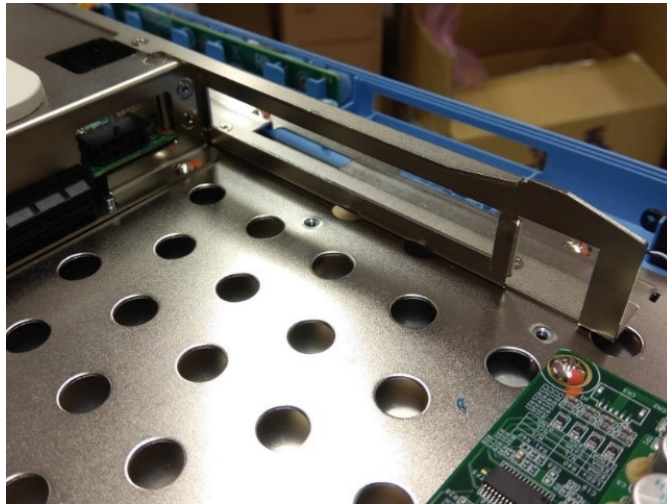

3. Install the PCIe (x4) card.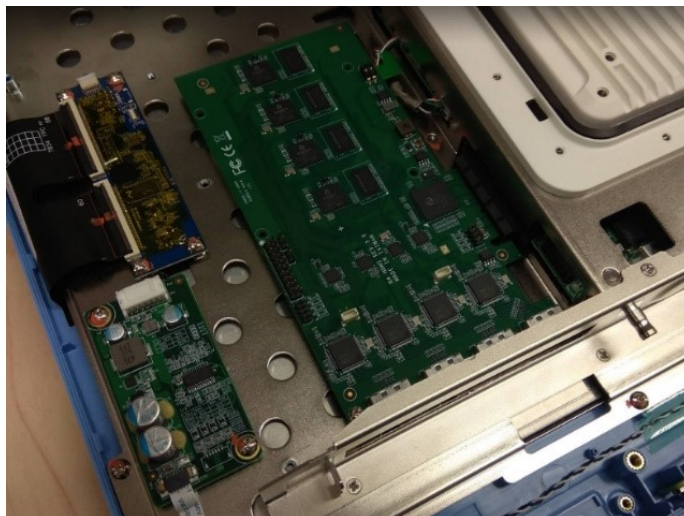

4. Install the fixed bracket with a screw and make sure the PCIe card is not moved.

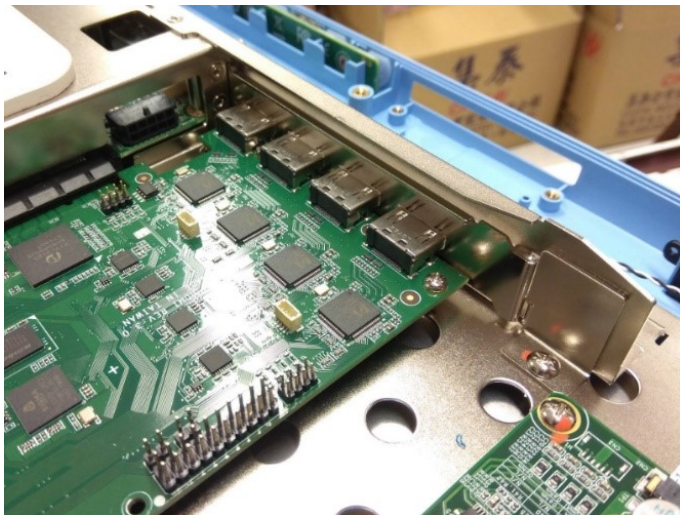# Chapter 12


## Hot-Swap Battery Pack Operation

The POC device supports a hot-swap battery pack. Through hot-swapping, a user can hotplug install two battery packs. This chapter describes how to install a battery.

## 12.1 Installing the Hot-Swap Battery Pack

1. Open the battery door:

Users can access the battery slot using the arrowhead buttons (circled in red below).

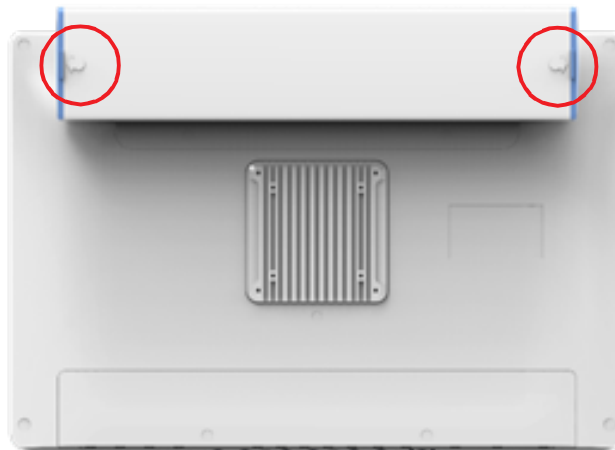1) Press the arrowhead button.

2) Pull open the door to access the battery slot.

**Caution!** We recommend holding the battery with both hands when inserting it into the POC-624-12 power system.

2. Insert battery pack:

The slot opening and battery packs feature notches (circled in red below) to indicate the correct installation direction.

**Caution!** Please hold the battery with both hands when installing the battery into the slot.

3. Close the battery door. When the door is fully closed, a “click” will be heard to ensure that the door is closed properly.


**Warning!** If not closed properly, the battery door may cause the battery to drop out.

**Caution!** Lay the pull ring and mylar (indicated in the image below) flat in their original positions before closing the door.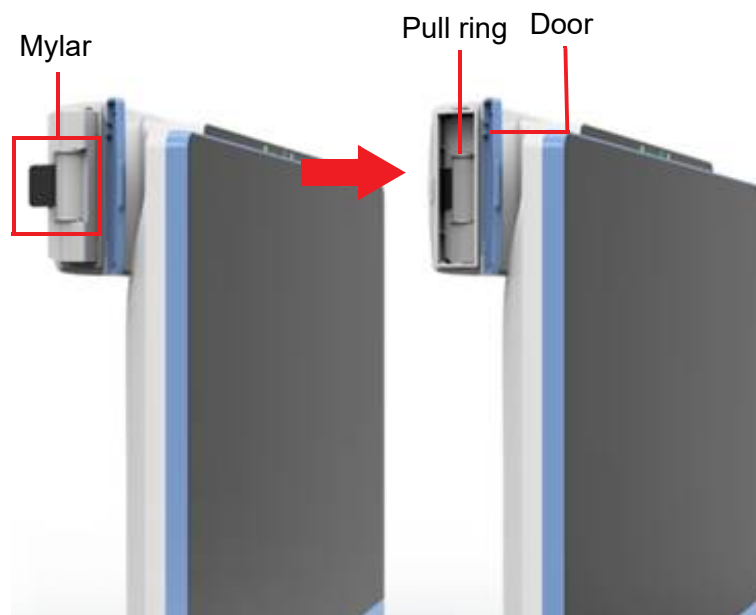

## 12.2 Battery Pack Capacity Indicator

The battery pack has a capacity indicator.

Press the button shown below circled in red. The battery pack will turn on the indicator to show its percentage of capacity.

The illuminated LED represents the battery capacity.

|                  | State-of-Charge      |
|------------------|----------------------|
| <b>LED Green</b> | <i>Current()</i> > 0 |
| LED1             | 0%~20%               |
| LED2             | 0~40%                |
| LED3             | 0~60%                |
| LED4             | 0~80%                |
| LED5             | 0~100%               |

## Hot-swap Battery Pack Battery Capacity Indicator and Windows Battery Information

1. The hot-swap battery pack has an LED indicator to show the battery's remaining capacity percentage.

In Discharge mode, the indicator shows the following battery levels.

**Table 12.1: Capacity**

| Battery Capacity | LED                           |
|------------------|-------------------------------|
| 0~10%            | Flash 1 LED (0.5Hz Frequency) |
| 11~20%           | 1 LED ON                      |
| 21~50%           | 2 LED ON                      |
| 51~100%          | 3 LED ON                      |

In Charge mode, the indicator shows the following battery levels.

**Table 12.2: Capacity**

| Battery Capacity | LED      |
|------------------|----------|
| 0~20%            | 1 LED ON |
| 21~50%           | 2 LED ON |
| 51~100%          | 3 LED ON |

These LED indicators are only ON when the POC system is turned ON. The LED indicator will turn off when the POC system is turned off. This setting can save battery life.

2. Windows Battery Icon  
The Windows battery icon can show each battery's power level percentage and the average level of dual batteries.

Windows will obtain each battery's level (unit: mAh) and calculate the average percentage.

Single battery capacity percentage =

$$\frac{\text{Battery remain capacity}}{\text{Battery full capacity}} * 100\%$$

Dual battery average capacity percentage =

$$\frac{\text{Battery 1 remain capacity} + \text{Battery 2 remain capacity}}{\text{Battery 1 full capacity} + \text{Battery 2 full capacity}} * 100\%$$

## 12.3 Battery Pack Regular Use

Below are instructions for maintaining a healthy battery:

1. Always fully charge new batteries before first use.
2. When the battery capacity is lower than the Windows preset low level (Default: 10%), Windows will display a low-battery notification. At this point, either replace the battery with a fully charged battery or plug in the power adapter to recharge the battery.
3. Avoid over-discharging a battery. Over-discharging will significantly reduce battery shelf life.
4. If the POC system is not in use, turn off the system to save battery power.
5. With a battery installed in either slot, the POC system can operate.

## 12.4 Battery Pack Storage

If the POC system or battery pack is stored in a warehouse for a long duration, the following tasks must be performed:

1. Always fully charge new batteries before moving them to a warehouse.
2. Before storing a battery and leaving it idle, recharge the battery to full capacity.
3. If the battery is left unused or idle for a long duration, perform a full recharge every 3 months.
4. Ensure that the POC system is switched off and double check that the power-on green LED indicator is off before storing the unit.

## 12.5 Battery Operation Sequence

When two batteries are installed, you should follow the below sequence to charge/discharge batteries.

1. Balanced mode (two batteries are of the same battery voltage):  
Charge or Discharge the two batteries together.
2. Un-balanced mode (two batteries are of different battery voltages):  
Charge: Charge the low-voltage battery first. When these two batteries reach the same voltage, charge the two batteries together.  
Discharge: Charge the high-voltage battery first. When these two batteries reach the same voltage, discharge the two batteries together.

*Enabling an Intelligent Planet*

## [www.advantech.com](http://www.advantech.com)

Please verify specifications before quoting. This guide is intended for reference purposes only.

All product specifications are subject to change without notice.

No part of this publication may be reproduced in any form or by any means, electronic, photocopying, recording or otherwise, without prior written permission of the publisher.

All brand and product names are trademarks or registered trademarks of their respective companies.

© Advantech Co., Ltd. 2023